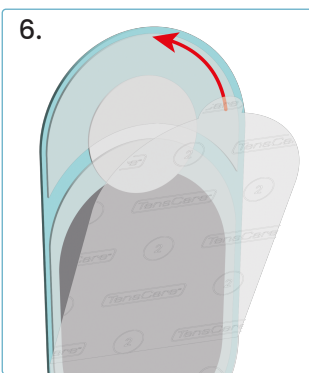
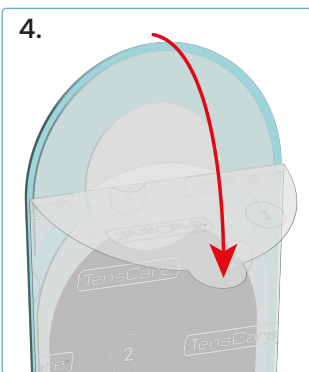
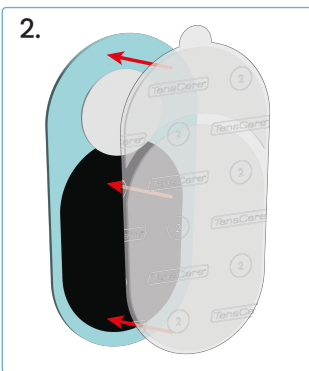
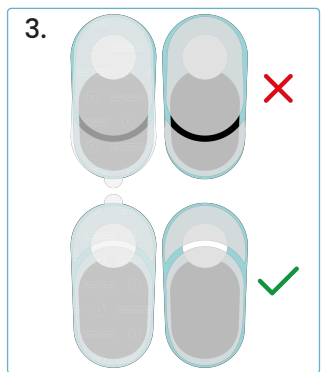
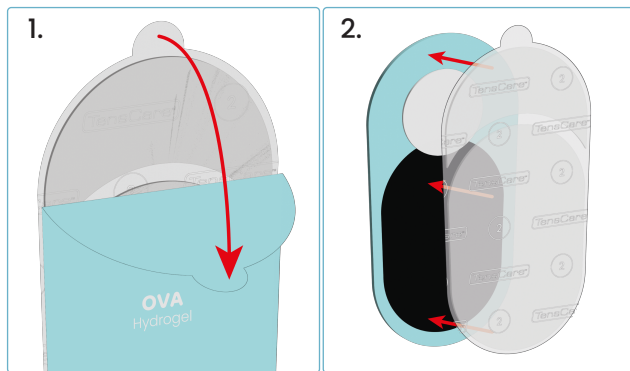
 Hydrogels must be applied to the electrodes in the correct orientation for the device to work effectively. Improper application can result in uneven or ineffective stimulation.

Before your session

1. It is essential to clean your skin before use.
2. Remove the hydrogels from the plastic bag.
3. Remove the coloured plastic liners first and apply to the bottom face of the electrodes. Take extra care to ensure that both sections of the hydrogel remain attached to the clear liner.
4. Remove the clear liners labelled 2 from the electrodes. Be careful to ensure both sections of the hydrogels remain stuck to the electrode.

-  **Note:** Ensure to align the Hydrogels in the correct orientation before applying to the bottom face of the electrodes.
-  **Note:** Please take care when removing liners from the hydrogels to ensure the gel does not get damaged.

5. Place the clear liner in the travel case. The coloured liner can be disposed of.
6. Place the OVA electrodes in desired position. (See electrode positioning options in the device manual.)

-  **Do not** ignore any allergic reaction to the hydrogels: If a skin irritation develops stop using Ova. A small number of users may have an adverse reaction to the gel.
-  **Do not** place hydrogels on skin which does not have normal sensation or on broken skin.

After your session

1. The stimulators will turn off automatically when your session is complete.
2. Remove the stimulators from the electrodes.
3. Remove the electrodes from the body by peeling by the outer edges. Be careful to ensure both parts of the hydrogel remain stuck to the electrode.
4. Re-apply the clear plastic liner onto the hydrogels and store the electrodes and the stimulators in the travel case. Proper storage between uses will help maximise the life of the gel.

Advice:

- The hydrogels are reusable and should provide 10–20 applications per gel. However, this will vary depending on the user's skin condition and how the hydrogels are stored between use. Try to limit the amount of time the gels are exposed to the air.
- Replace the hydrogels when they lose their stickiness. Poor connection to the body may cause ineffective treatment, discomfort and skin irritation.
- The hydrogels supplied are for single patient use. The adhesive is a peelable hydrogel (water based).
- The shelf life of an unopened packet of hydrogels is 3 years. This can be affected by very high temperatures or very low humidity.

Important: Observe all the Contraindications, Cautions and Warnings included with the Ova device manual.

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TensCare Ltd. 9 Blenheim Rd, Epsom, Surrey, KT19 9BE, U.K. +44 (0)1752 723 434
Advena Ltd, Tower Business Centre, 2nd Flr, Tower Street, Swatar, BKR 4013, Malta